

EWC Lift "Quick Reference"

(Refer to sling manufacturer's instructions as needed)

1. Evaluate the patient.
2. Visually inspect the lift cord, and proper lift operation.
3. Put sling on the patient.
4. Move T-bar to proper location and attach sling.
5. Raise the T-bar until slack is removed from the straps.
6. **Stop lifting.**
7. Make sure patient's hands and feet are clear for lifting.
Also make sure sling and straps are not hooked on any obstructions.
8. Lift patient and move to desired location.


EWC Lift Systems Ltd.

360-666-3565

Web Site www.ewclift.com



T-bar
WITHOUT
scale attached.
(Take off clip
at bottom of
scale).

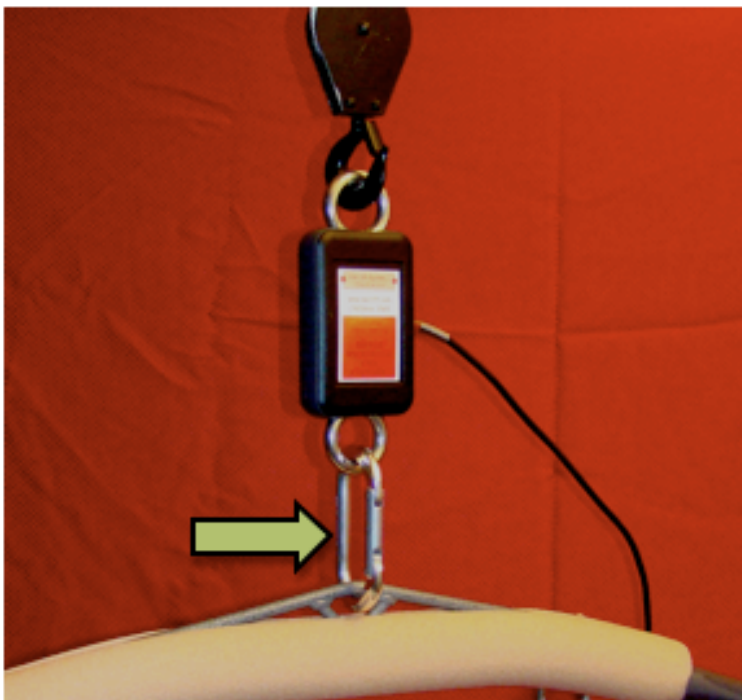

T-bar WITH
scale attached.
(Clip on bottom
is to join scale
and t-bar).